


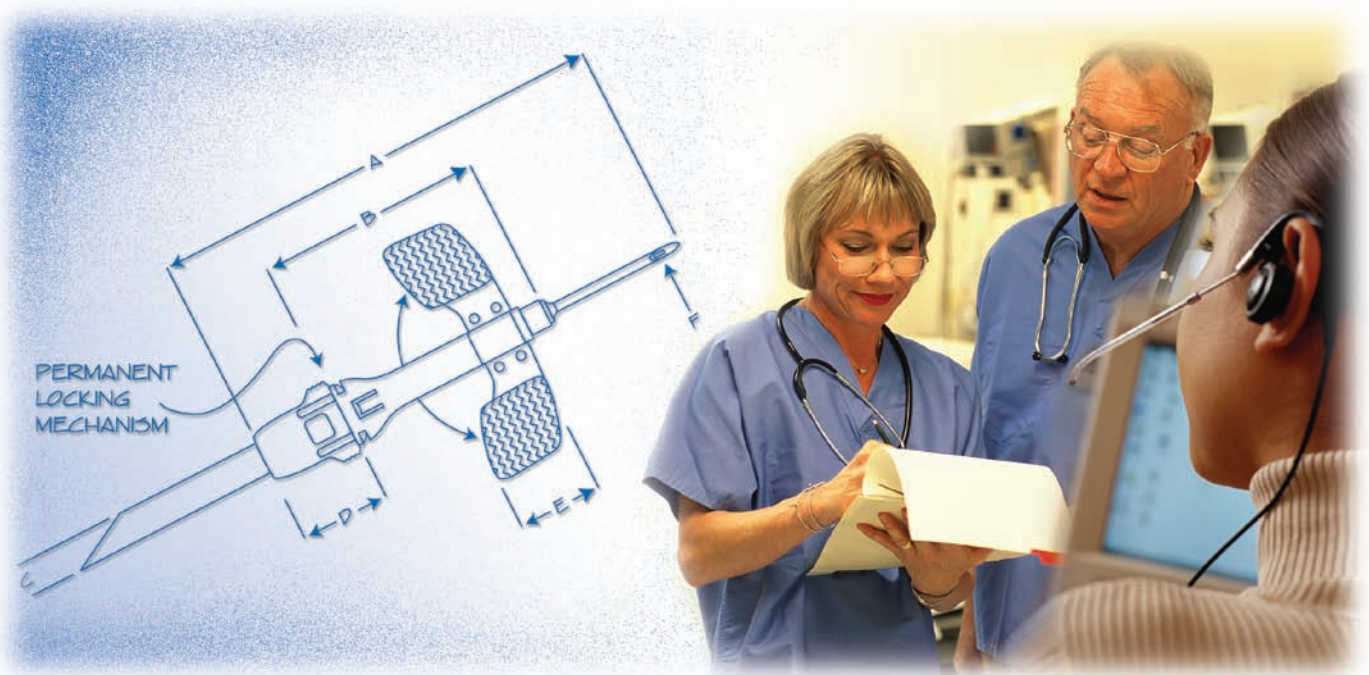

D I A L Y S I S P R O D U C T S

Engineered for safety.

Supported by service.



Next Generation

SAFETOUCH™ II

SAFETY FISTULA NEEDLE



Once locked, stays locked.

- The only system with *built-in* safety advantages: a permanent locking feature eliminates the worry of accidental needlesticks; textured wings provide secure grip; system is low profile and is designed to meet all OSHA safety requirements.



Quality you expect. Support you deserve.

- The NIPRO CONNECTSM PROGRAM is our commitment to provide every dialysis customer with products of unmatched value through a continuous quality improvement process. In addition, it assures that service, education and the required technical support are provided to each of our business partners.



Next Generation


SAFETOUCH™ II

SAFETY FISTULA NEEDLE

Below are highlights of proper use of the SafeTouch™ II Needle. For complete information, please refer to the Instructions For Use printed on the outer product carton.



Peel open the package from the position marked "PEEL HERE." Do not break open the package to remove product.

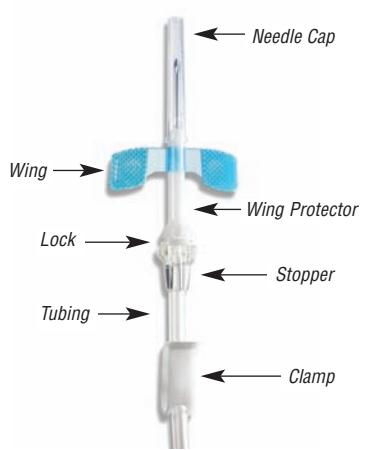

Safety is built in to the SafeTouch™ II needle design. After removal from package, check to ensure that wing protector is securely locked into the stopper.



At end of procedure, close clamp, apply pressure to site and hold down wing protector; release lock by pressing both sides of stopper, sliding stopper back until it becomes fixed with audible or tactile click.



Once wing protector is fully engaged, dispose of protected SafeTouch™ II needle in an approved sharps container.

ORDER SPECIFICATIONS

Product Code	Description	Wing Color
FS+142530BC	14G x 1"	Purple
FS+142530TP	14G x 1"	Purple
FS+152530BC	15G x 1"	Blue
FS+153230BC	15G x 1 1/4"	Blue
FS+152530TP	15G x 1"	Blue

Product Code	Description	Wing Color
FS+162530BC	16G x 1"	Green
FS+163230BC	16G x 1 1/4"	Green
FS+162530TP	16G x 1"	Green
FS+172530BC	17G x 1"	Orange
FS+173230BC	17G x 1 1/4"	Orange
FS+172530TP	17G x 1"	Orange

All of the above Product Codes include 12" tubing, Fixed Wing and Backeye Needle. If preferred, any of these can also be ordered with Rotating Wing, 6" tubing, and/or without Backeye Needle. TP code contains two needles per pack.

Please consult product labels and inserts for indications, contraindications, hazards, warnings, cautions and directions for use.



NIPRO MEDICAL CORPORATION
 3150 107 Avenue
 Miami, FL 33172
 Tel: (305) 599-7174
 Fax: (305) 599-8454

For additional information on this product and/or the NIPRO CONNECT™ PROGRAM, call us toll-free at 1-888-647-7698/ext. 405.

© 2004 Nipro Medical Corp. Printed in U.S.A.
 SafeTouch™ II is a trademark of Nipro Medical Corp.
 0002R APA 3/05 2M